

Cleaning Your Splint Bath



Frequency – Two or more times monthly. Strava recommends each facility determine its cleaning frequency based on usage, water grade, visual inspection, and sanitation protocols that apply to your facility and/or JCAHO.



1 TURN UNIT OFF FROM THE SWITCH ON THE BACK OF THE UNIT AND **EVACUATE WATER**

See [User Manual](#) procedure.



2 ADD WATER & CLEANING SOLUTION

Fill unit just above the water sensor with a 2:1 ratio of tap water to cleaning solution.



Cleaning Solution = Vinegar -OR- Coffee Machine Commercial Descaler



3 TURN THE UNIT BACK ON and leave it running for 20-30 minutes to allow the water/cleaning solution mixture to heat.



4 EVACUATE WATER

Wait 10 minutes then evacuate water. See [User Manual](#) procedure.



5 TURN UNIT OFF

Unplug and allow unit cool down before proceeding.



6 CLEAN BI-FOLD LID & INSIDE WATER BASIN

Using a Scotchbrite® Scour Pad or cloth, wipe the inside tub with warm water. To remove scum, waterlines, mineral deposits, etc., add a mild detergent. Be sure to rinse thoroughly and wipe dry. **Do NOT use bleach, ammonia-based cleaning products, or extra abrasive products such as steel wool.**




7 REFILL WITH WATER

Fill unit to operating level with tap water. See [flip side](#) or [User Manual](#) if you use distilled water.



8 TURN UNIT ON

 Repeat procedures if necessary

**See Flip Side For Annual
Maintenance Recommendations**



QUICK TIPS

DISTILLED WATER

1 Cup Tap Water



- PER -



1 Gal. Distilled

Tap water is required for the low water sensor to work properly.

Follow these proportions if you are in a hard-water environment and require distilled water.

WATER EVACUATION

- Turn unit off using the power switch at the back.
- Allow the water to cool before water evacuation.
- Be sure that the supplied drain hose is properly inserted at the back the unit.
- Turn the quarter turn valve counter-clockwise to release the water.



ABOUT US

The name Pivotal Health Solutions is renowned for its cutting-edge, well-crafted, and ergonomically designed products. We foster a culture of creativity throughout our organization, from our innovative products to our dedicated employees, which distinguishes us from competitors. As a prominent manufacturer in the industry, we specialize in a wide array of healthcare and wellness products. At Pivotal Health Solutions, we depend on the collaborative efforts of our employees, customers, dealers, and vendors to consistently deliver valuable solutions that keep us ahead of the curve.



www.pivotalhealthsolutions.com | 800.743.7738

FOR TECHNICAL SERVICE

Visit www.stravasolutions.com and submit a Service Inquiry



Refer to the user manual at: www.stravasolutions.com