



Technical Specifications

- Overall Height.....22"
- Overall Width.....24"
- Overall Length.....72"
- Weight.....70 lbs (31 kg)
- Weight Capacity.....600 lbs (272 kg)

Product Description **MD**

Intended Use: The EB8000 bench is intended for use in clinical settings to support patients during chiropractic treatments.

Warnings and Precautions






- Do not exceed table's maximum weight capacity of 600 lbs (272 kg).
- Periodically recheck tightness of all nuts, bolts and screws.
- This table is NOT intended for use with other accessories.
- Use only Pivotal Health Solutions replacement parts.
- Contact Pivotal Health Solutions if service is needed (1-800-743-7738).







Tools needed for Assembly

- Two 1/2" Wrenches
- One 3/8" Wrench

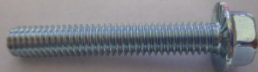If you have trouble understanding these instructions contact your dealer or Pivotal Health Solutions support at 800-743-7738.

Before beginning assembly, verify you have the ALL fasteners and parts listed below:

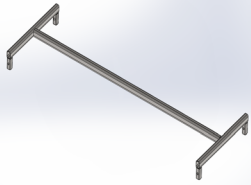
PART		QUANTITY INCLUDED	DESCRIPTION	PHS REPLACEMENT #
A		8X	5/16-18 Nylon Insert Lock Nut	87561
B		8X	5/16-18 x 2-1/2 Hex Head Bolt	66224
C		16X	1/4-20 x 1-3/4 Serrated Bolt	60822
D		2X	1/4-20 x 1-1/2 Hex Head Bolt	60440
E		2X	1/4-20 Nylon Insert Thin Locknut	95769

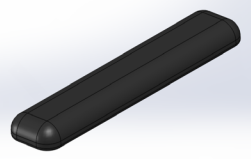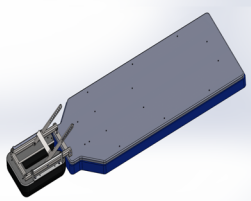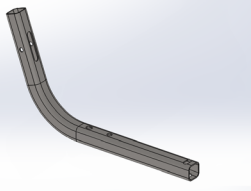PART		QUANTITY INCLUDED	PART DESCRIPTION	PHS REPLACEMENT #
1		1X	Bottom Brace	81006
2		2X	BENCH LEG	81005
3		2X	ARM REST	81008
4		2X	ARM CUSHION	81020
5		1X	TOP CUSHION ASSEMBLY	
6		2X	HEAD LOCK TUBE	81013

Assembly STEP 1: Attach Bottom Brace **1** to Bench Legs **2**^{2X} with 2-½" Bolt **B**^{4X} and Lock Nut **A**^{4X}.

NOTE: Fastener **A** goes to inside of frame. (See Detail 2.1).

NOTE: Ensure arm rest holes in bench legs face the same direction (See Detail 2.2).

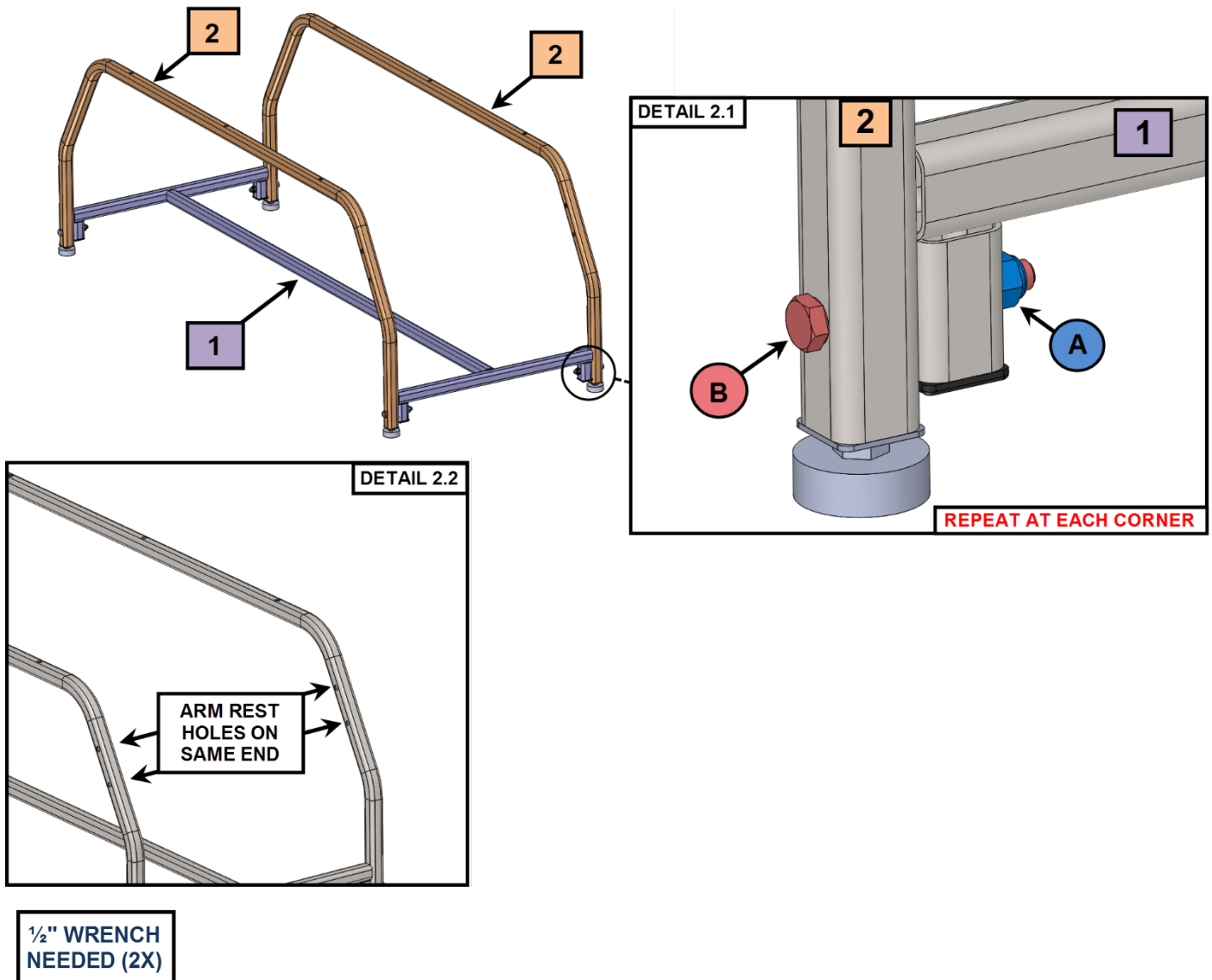


FOR EASE OF ASSEMBLY, LEAVE ALL HARDWARE LOOSE UNTIL COMPLETELY ASSEMBLED.

Assembly STEP 2: Attach arm rest **3**^{2X} to base frame with 2-½" Bolt **B**^{4X} and Lock Nut **A**^{4X}.

NOTE: Fastener **A** goes to inside of frame. (See Detail 2.3).


NOTE: Arm rests should angle downward.

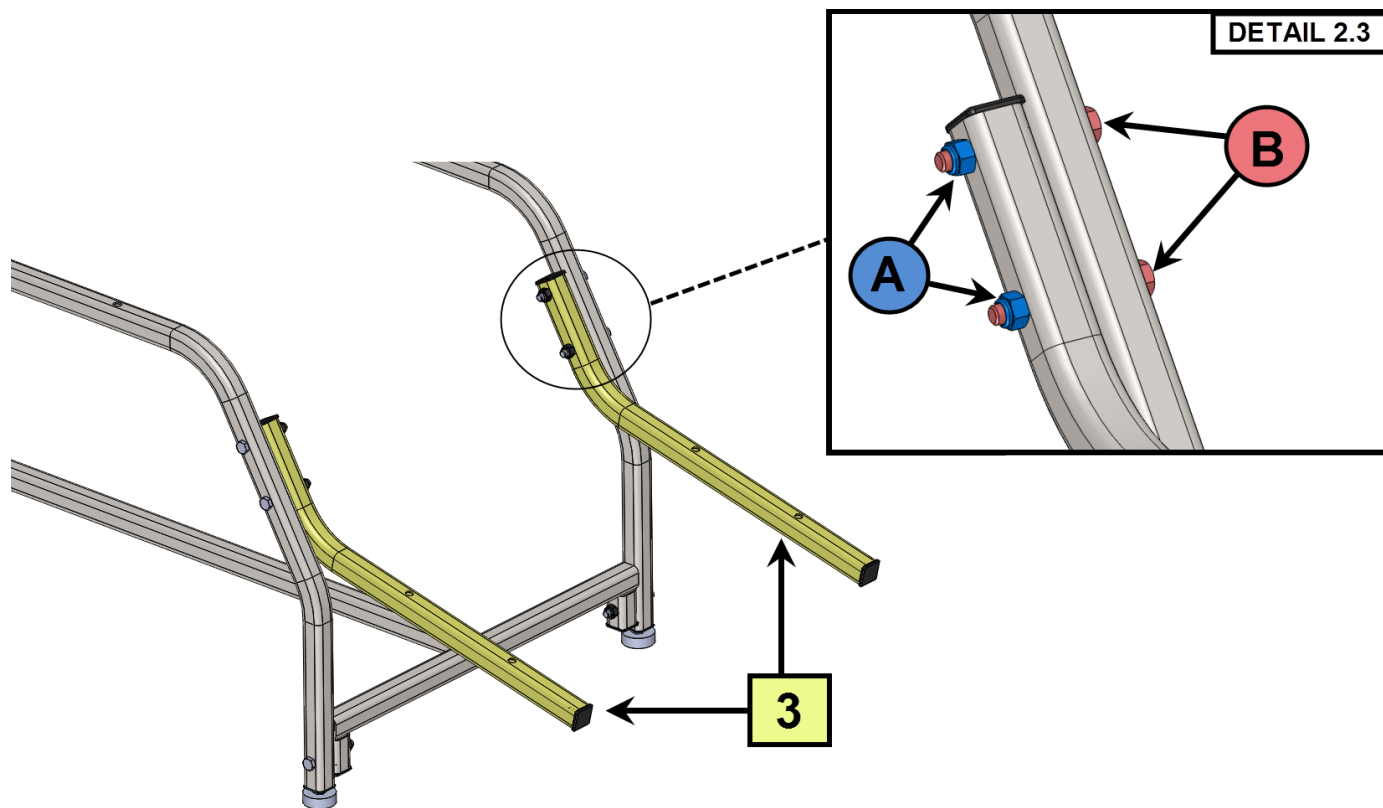

**½" WRENCH
NEEDED (2X)**

FOR EASE OF ASSEMBLY, LEAVE ALL HARDWARE LOOSE UNTIL COMPLETELY ASSEMBLED.

Assembly STEP 3: Attach Arm Cushions **4**^{2X} to arm rest with 1-3/4" Bolt **C**^{4X}
(See Detail 2.4).

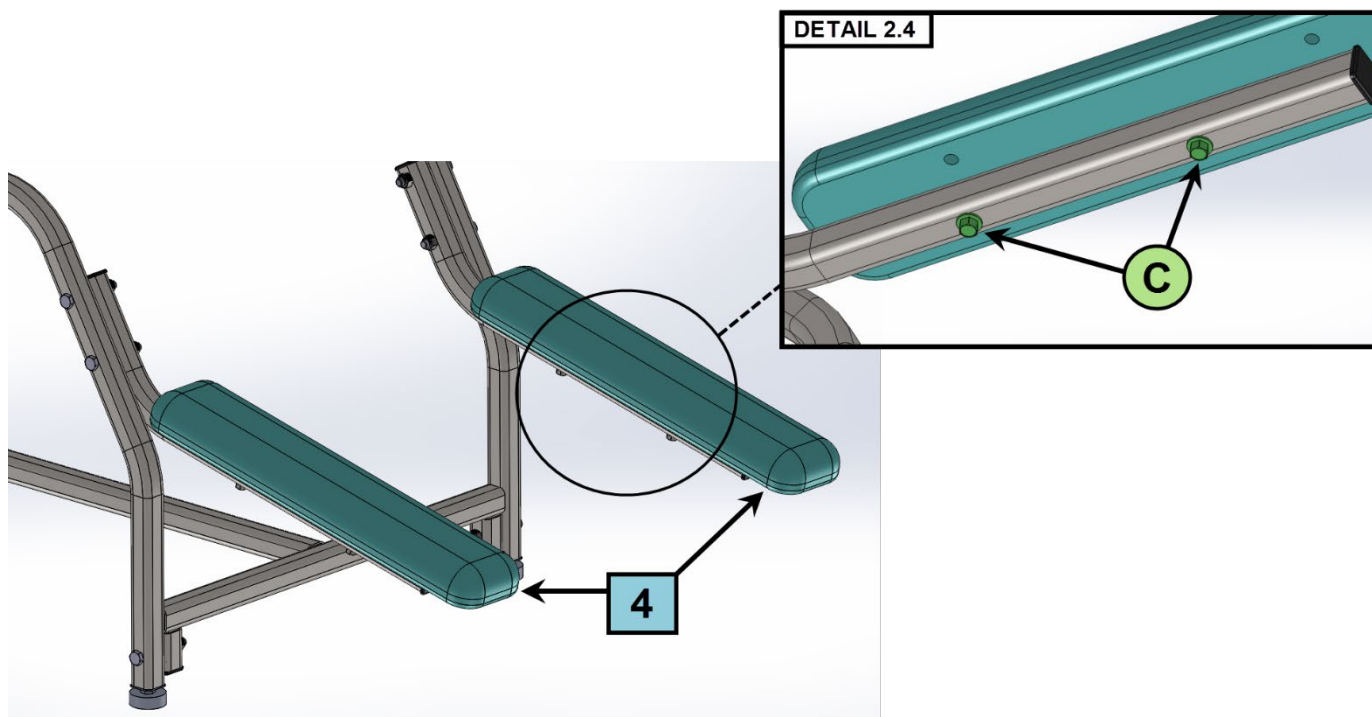

**3/8" WRENCH
NEEDED**


**FOR EASE OF ASSEMBLY, LEAVE ALL HARDWARE LOOSE UNTIL COMPLETELY
ASSEMBLED.**

Assembly STEP 4: Slide head support bracket through slots in Head Lock

Tube **6**^{2x}.

NOTE: Attach Head Lock Tubes to Top Cushion Assembly **5** using 1-

3/4" Bolt **C**^{6x}.

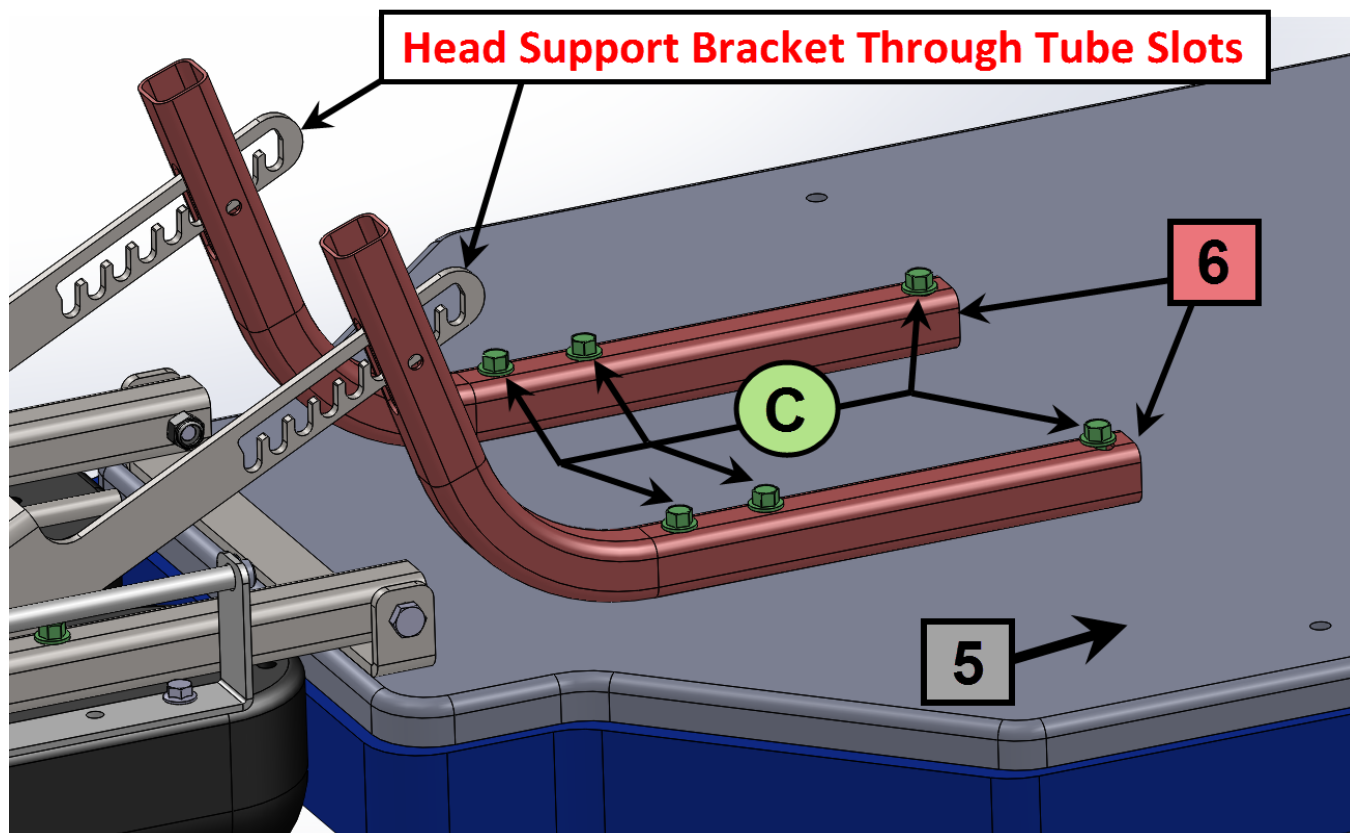


FOR EASE OF ASSEMBLY, LEAVE ALL HARDWARE LOOSE UNTIL COMPLETELY ASSEMBLED.

Assembly STEP 5: Attach Head Support Brackets to Head Lock Tubes with

$\frac{1}{4}$ -20x1- $\frac{1}{2}$ Bolt **D**^{2X} and $\frac{1}{4}$ -20 Locknut **E**^{2X}.

NOTE: Locknuts go to inside of tubes. (See Detail 2.5)

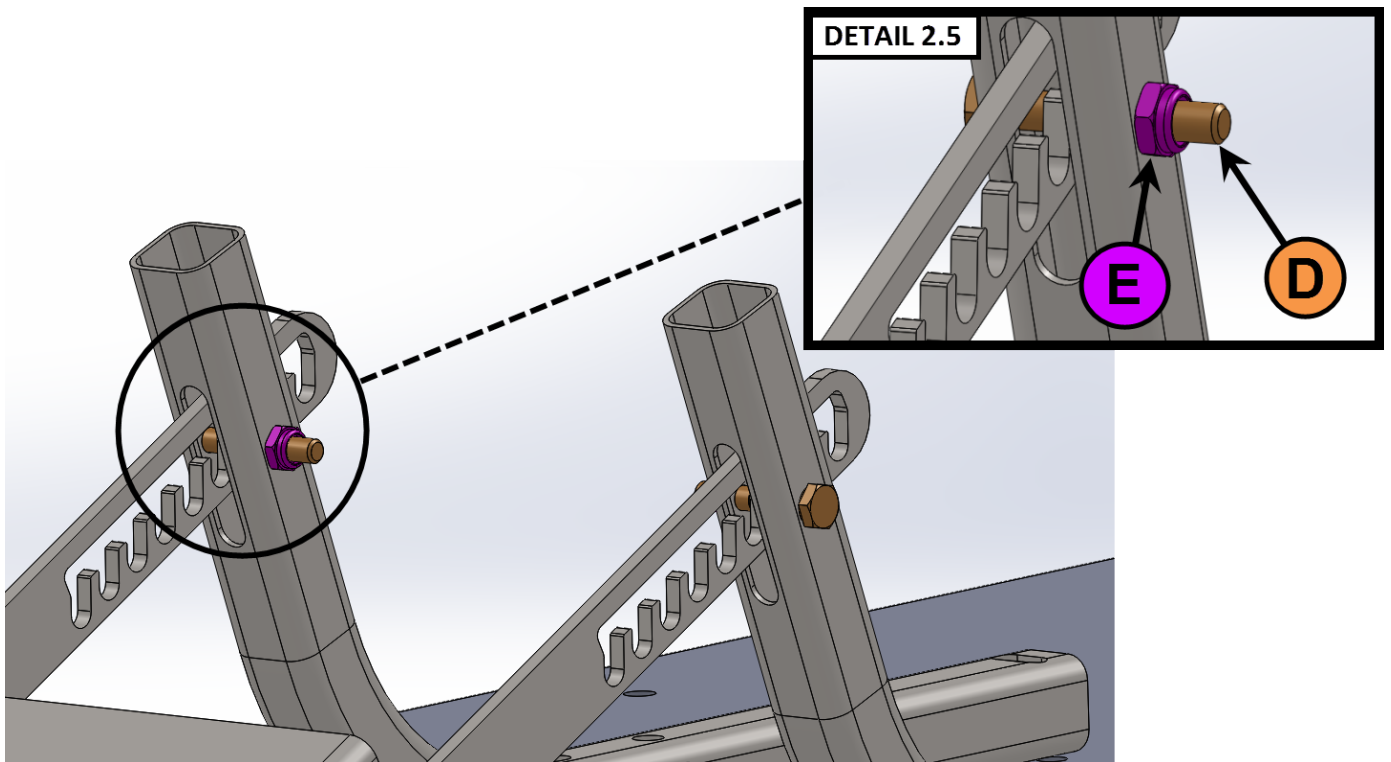


FOR EASE OF ASSEMBLY, LEAVE ALL HARDWARE LOOSE UNTIL COMPLETELY ASSEMBLED.

Assembly STEP 6: Attach Top Cushion Assembly to Base Frame with 1-

3/4" Bolt ^{6X}.

3.6.1 The Top Cushion fastens to the frame at six locations.

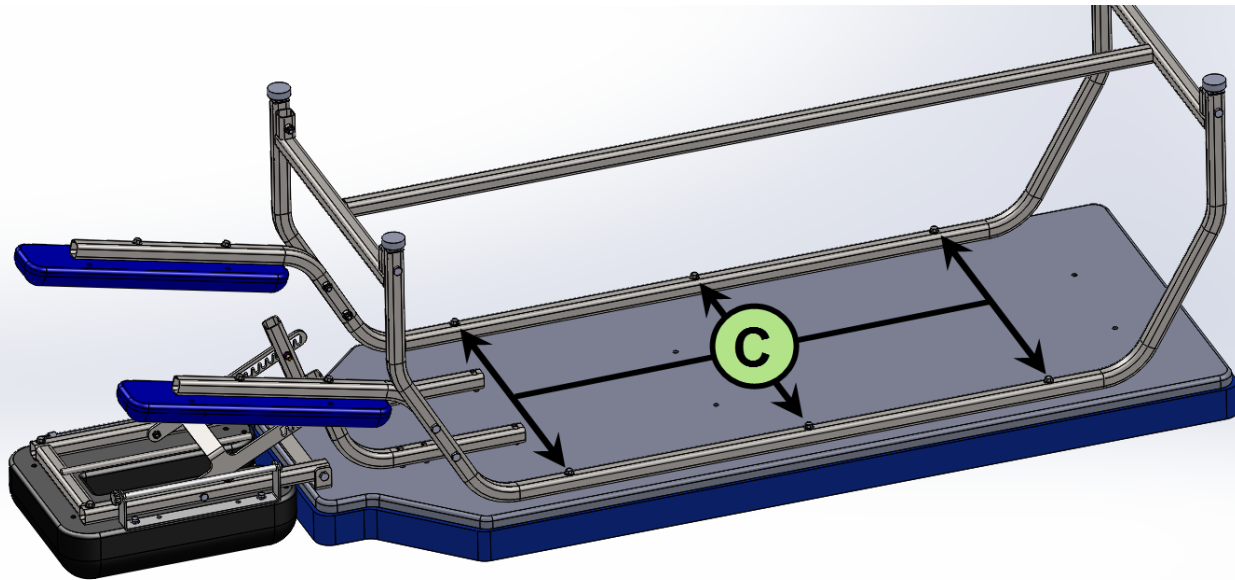

**3/8" WRENCH
NEEDED**



**ASSEMBLY IS COMPLETE. GO BACK AND TIGHTEN ALL
HARDWARE ON TABLE BEFORE USE.**

Upholstery Cleaning and Care

For light soiling, a solution of 10% household liquid dish soap with warm water, applied with soft damp cloth. Rinse with clean water and dry.

- For heavy soiling, dampen a soft white cloth in a one to one (1:1) solution of Fantastik® and water OR Formula 409® and water. Rub gently and rinse with a water dampened cloth.
- For more difficult stains, dampen a soft white cloth with a solution of household bleach (10% bleach / 90% water). Rub gently and rinse with a water dampened cloth to remove bleach concentration.
- DO NOT USE ALCOHOL BASED CLEANING AGENTS!
- IMPROPER CLEANING WILL CANCEL WARRANTY!

This information is not a guarantee and does not relieve the user from their responsibility of the proper and safe use of the product and all cleaning agents. The use of certain agents can be harmful to the surface appearance and lifespan of vinyl. Pivotal Health Solutions assumes no responsibility resulting from the use of such agents to the vinyl.

Service, Warranty, Manufacturer Contact Information

If a serious incident occurs in relation to this device, report the incident to Pivotal Health Solutions with the contact information below.

If the incident occurs in the EU, also report any serious incident to the competent authority of the Member State in which the user and/or patient is established.

If service is required, contact Pivotal Health Solutions with the contact information below or email service@pivotalhealthsolutions.com.

 Pivotal Health Solutions, Inc. 3003 9 th Ave. SW; Watertown, SD, 57201 USA
 800-743-7738  011-1-605-753-0110  605-882-8398
 http://www.pivotalhealthsolutions.com



Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands



MedEnvoy
Gotthardstrasse 28
6302 Zug
Switzerland

Rev F 07/17/2023